

### IMPORTANT SAFETY INFORMATION: PLEASE READ

- ⚠ **WARNING:** Important! Read and understand these instructions before using the walker. If you do not understand any part of these warnings, cautions or instructions, contact a healthcare professional for direction in the use of this product. If the walker is not properly used, personal injury and damage to the walker could result.
- ⚠ **WARNING:** If components are damaged or missing, contact your dealer immediately. **DO NOT** use substitute parts. The use of non-Gendron replacement parts will void the warranty, and could create a hazardous condition resulting in serious personal injury.
- ⚠ **WARNING:** This product has a maximum weight capacity of 750 lb, **EVENLY DISTRIBUTED**; and when using the wheel kit, 650 lb, **EVENLY DISTRIBUTED**.
- ⚠ **WARNING:** Cancer and Reproductive Harm - [www.p65warnings.ca.gov](http://www.p65warnings.ca.gov).
- ⚠ **WARNING:** GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper assembly, installation or use of this product.

### INTENDED USE

The intended use of this product is as a walking aid, manipulated by both arms. Recommended user height is approximately 5'2" – 6'4" (157 cm – 193 cm).

### UNPACKING

Check for any obvious damage to the carton or its contents. If damage is evident, **DO NOT USE**. Contact carrier / distributor for further instruction.

### OPEN AND CLOSE WALKER

**To open walker:** Grasp one handgrip and pull away from center of walker. Repeat with other handgrip. Push handgrips away from each other until both walker release mechanisms click into place.

**To close walker for transport or storage:** Push downward on release mechanisms and hold. Grab the nearest handgrip and push toward center of walker. Repeat for other handgrip. Walker is ready for transport and storage after folding.

### ADJUST WALKER HEIGHT

*Info: Consult your healthcare professional to determine appropriate walker height.*

1. Adjust walker height so your arms, relaxed at your side, meet the walker handgrips at wrist height. While your hands are on the hand grips, your arms should flex 20° to 30°.
2. To adjust, press the spring button on each leg, allowing the leg to move into the desired position.
3. Once all legs are adjusted to the same height, ensure the spring button is secure in each hole.

*Info: To install wheel kit on walker two front legs **only**: press spring button, twist leg slightly, and pull the leg out of the frame. Insert wheel kit tube into front frame tube. **Ensure spring button is secure.***


- ⚠ **WARNING:** Before use, ensure all four legs are adjusted to the same height, all four spring buttons protrude completely from holes, and any attachments are securely locked in position.

### OPERATION – WALKING WITH YOUR WALKER

- ⚠ **WARNING:** Avoid operation in unsafe walker conditions, such as soft or slippery surfaces, uneven terrain, in pools, showers or other wet environments; this could cause damage to the walker and possible personal injury to the user.

- ⚠ **WARNING:** Do not step on or apply pressure to walker crossbars; this could damage the walker and cause personal injury.

1. Start with both feet even with the walker's back legs; stand behind your walker with walker frame facing forward and a hand placed securely on each handgrip.
2. Carefully pick up the walker and move it in front of you; ensure all four walker legs are on the floor; step forward one foot at a time, and continue.

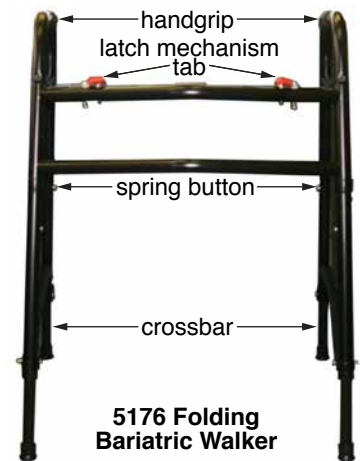

## CLEANING AND MAINTENANCE

**⚠ WARNING: Before each use, ensure all hardware is tight, latch mechanisms tabs are locked in place, and legs are securely attached in appropriate height position.**

**▲ NOTICE: DO NOT use solvents, abrasive cleaners, or scouring pads to clean this product.**

To remove dust and dirt, wipe the walker gently with a clean, soft, damp cloth. To clean it, use a clean, soft cloth dampened with a mild soap and water solution, rinse with clear water, then dry with a soft cloth. Check spring buttons and attachments for tightness. Replace tips as needed.

## LIMITED WARRANTY

### SCOPE OF WARRANTY

GF Health Products, Inc. ("GF") warrants to the original purchaser only that it will replace or repair components, at GF's sole discretion, that are defective in material or workmanship under normal use and service. All warranties are conditioned upon the proper use of the products strictly in accordance with good commercial practice and applicable GF instructions and manuals, including proper use and maintenance. To the extent that a component is warranted by a third party, GF conveys all of its rights under that warranty to the original purchaser, to the extent permitted. This limited warranty shall only apply to defects that are reported to GF's customer service team within the applicable warranty period and which, upon examination by GF or its authorized representative, prove to be a warranty item. This limited warranty is not transferable. Within the guidelines set forth in this document, this product is warranted for three (3) years. The applicable warranty period shall commence from date of shipment to the original customer, unless there is an expiration date on the component in which case the warranty shall expire on the earlier of warranty period or the expiration date.

### OBTAINING WARRANTY SERVICE

This limited warranty shall only apply to defects that are reported to the Distributor from whom the Customer purchased the product within the applicable warranty period. If there is not a Distributor, you must contact GF directly by calling 1.770.368.4700, sending a fax request to 1.770.368.2386, or by e-mailing a request to [cs@grahamfield.com](mailto:cs@grahamfield.com). Specific directions will be provided by the Customer Service Representative. Failure to abide by the specific directions will result in denial of the warranty claim.

### EXCLUSIONS

The warranty does not cover and GF shall not be liable for the following:

- 1) Defects, damage, or other conditions caused, in whole or in part, by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner;
- 2) Products which are not installed, used, or properly cleaned and maintained as required in the official manual for the applicable product;
- 3) Products considered to be of a non-durable nature including, but not limited to: tips, casters, filters, fuses, gaskets, lubricants, and charts;
- 4) Accessories or parts not provided by GF;
- 5) Charges by anyone for adjustments, repairs, replacement parts, installation or other work performed upon or in connection with such products which are not expressly authorized in writing, in advance, by GF;
- 6) Any labor or shipping charges incurred in the replacement part installation or repair;
- 7) Costs and expenses of regular maintenance and cleaning; and
- 8) Representations and warranties made by any person or entity other than GF.

### ENTIRE WARRANTY, EXCLUSIVE REMEDY AND CONSEQUENTIAL DAMAGES DISCLAIMER

THIS WARRANTY IS GF'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. GF MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. IF ANY MODEL OR SAMPLE WAS SHOWN TO THE CUSTOMER, SUCH MODEL OR SAMPLE WAS USED MERELY TO ILLUSTRATE THE GENERAL TYPE AND QUALITY OF THE PRODUCT AND NOT TO REPRESENT THAT THE PRODUCT WOULD NECESSARILY CONFORM TO THE MODEL OR SAMPLE IN ALL RESPECTS. THIS WARRANTY IS LIMITED TO THE REPAIR OR REPLACEMENT OF THE DEFECTIVE PARTS. GF SHALL NOT BE LIABLE FOR AND HEREBY DISCLAIMS ANY DIRECT, SPECIAL, INDIRECT, INCIDENTAL, EXEMPLARY OR CONSEQUENTIAL DAMAGES, INCLUDING, BUT NOT LIMITED TO: DAMAGES FOR LOSS OF PROFITS OR INCOME, LOSS OF USE, DOWNTIME, COVER, OR EMPLOYEE OR INDEPENDENT CONTRACTOR WAGES, PAYMENTS AND BENEFITS.

The warranties contained herein contain all the representations and warranties with respect to the subject matter of this document, and supersede all prior negotiations, agreements and understandings with respect thereto. The recipient of this document hereby acknowledges and represents that it has not relied on any representation, assertion, guarantee, warranty, collateral contract or other assurance, except those set out in this document.

For additional information on this product or this warranty, please contact a GF Customer Service Representative.

### NOTES:

- 1) Additional terms and conditions may apply.
- 2) Freight claims must be notated on the appropriate shipping documents and must be made with immediacy. International, federal and state regulations govern specific requirements for freight claims. Failure to abide by those regulations may result in a denial of the freight claim. GF will assist you in filing the freight claim.
- 3) Claims for any short shipment must be made within three (3) days of the invoice date.



1.770.368.4700

Information contained herein is subject to change.  
The most current and complete product information can be found on our website.  
[www.grahamfield.com](http://www.grahamfield.com)



© 2020, GF Health Products, Inc. All Rights Reserved.  
Gendron is a trademark of GF Health Products, Inc.  
GF Health Products, Inc. is an ISO 13485:2016 Certified Company.  
Manufactured for GF Health Products, Inc.,  
One Graham-Field Way, Atlanta GA 30340-3140  
Made in China