

- ⚠ **WARNING: Important! Read and understand these instructions before installing the scale. If you do not understand any part of these warnings, cautions or instructions, contact a healthcare professional for direction in the use of this product. If the scale is not properly installed, personal injury and damage to the scale could result.**
- ⚠ **WARNING: If components are damaged or missing, contact your GF authorized distributor immediately. DO NOT use substitute parts. The use of non-Gendron replacement parts will void the warranty, and could create a hazardous condition resulting in serious personal injury.**
- ⚠ **WARNING: Cancer and Reproductive Harm - www.p65warnings.ca.gov.**
- ⚠ **WARNING: GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper assembly, installation or use of this product.**

INSTALLATION INSTRUCTIONS

- ⚠ **WARNING: Perform this installation only while patient lift is unoccupied.**

Components required for installation: G27085 scale, shoulder bolt, and nut

Tools required: 3/16" hex key and socket wrench with 3/8" socket



step 1



step 2



step 3

1. Remove the T-handle pin and spacers, enclosed in red ellipse above, from the spreader bar.
2. Use hex key and socket wrench to attach scale to spreader bar with shoulder bolt and nut, as shown above.
3. Attach scale to the lifting point using the quick release pin, as shown above.

- ⚠ **WARNING: Ensure spreader bar and scale are securely installed before using patient lift.**



1.770.368.4700

Information contained herein is subject to change.
The most current and complete product information can be found on our website.
www.gendroninc.com / www.grahamfield.com



© 2021, GF Health Products, Inc. All Rights Reserved.
Gendron is a trademark of GF Health Products, Inc.
GF Health Products, Inc. is an ISO 13485:2016 / MDSAP Certified Company.
Manufactured for GF Health Products, Inc.,
One Graham-Field Way, Atlanta GA 30340-3140